



The functions of the smartwatch

Table of Contents

1. Related parts	p.3
2. Buttons and settings	p.3
3. Start the smartwatch	p.4
4. To charge the smartwatch	p.4
5. How to press the alarm?	p.6
6. To receive calls to the watch	p.7
7. Changing bracelets	p.7
8. Specifications and recycling	p.8
9. Contact information	p.8

1. Related parts



Picture 1

1. Smartwatch Bo



Picture 2

2. Charging cord



Picture 3

3. Transformer

2. Buttons and settings



Picture 4

Safety in everyday life!



The knowledge that someone always receives the alarm and knows the user's GPS position, and thus can act (relatives or Sensorem AB) should provide security - both for the user and relatives. If you want to change the number of relatives and/or their order, please contact Sensorem's support.

3. Start the smartwatch

- The watch is pre-programmed and ready to use immediately upon delivery. Start the watch by pressing the alarm button on the right side of the watch (see picture 4) until the Sensorem logo appears on the screen (around 5 seconds).
- Once turned on, the watch's screen is black in standby mode. The screen is lightened up by a short press on the alarm button, the watch then shows the time with an analogue clock (there is no risk of accidentally setting off the alarm as the alarm button needs to be held down for three seconds for an alarm to occur). By dragging your finger to the left on the screen, the watch's main menu appears (picture 4). To see the clock in analog time again, swipe right. The screen always turns off automatically after 15 seconds.

4. To charge the smartwatch

- The smartwatch is charged by placing the metal contacts on the associated charger, on the back of the watch (picture 2). The metal contacts are automatically drawn towards the watch. If you place the charger in the wrong direction, the charging connector on the watch will push the charger away. When the watch is charging, it says "laddar" (charging) on the screen.
- The battery life of the watch is 2-3 days, depending on the usage. Charging the watch takes about 3 hours and it is possible to activate the alarm while the watch is charging. When the watch is fully charged, it says "fulladdat" (fully charged) on the screen.
- When there is 25% of the battery left, the watch lights up with the text "**Påminelse, låg batterinivå**" (Reminder, low battery). At the same time a voice message is played saying "*low battery, please charge your watch.*" (The watch says the message in swedish.) The message remains on the screen until the user confirms that the message has been seen by pressing the alarmbutton once. If the low battery level occurs between 23:00 and 6:00 of the day, the watch will wait to play the message until the time interval has passed. If the watch is not charged after this, the screen will continue to display the text "**Lågt batteri**" (low battery) until the watch is completely discharged or recharged.



Picture 5

- When you connect the charger, the watch displays a box saying "Laddar" (charging) and when the watch is fully charged, the screen displays a box saying "Fulladdat" (fully charged).

- You can charge the watch regardless of its battery level.
- When charging for more than 3 hours, it is not necessary to wait until it says "Fulladdat" (fully charged) on the screen. The watch is optimized to delay the charging time from 95-100%, as this extends the life of the battery.

If the watch discharges

- If the watch discharges completely and is turned off, it must be turned on again in order to work. To do this, hold down the alarm button until you see the Sensorem logo lightening up. When doing this, there is no risk of accidentally setting off an alarm, so don't be afraid to press the button to start the watch again.
- To see if the watch is switched off/discharged, you can press the alarm button once and see if the screen lights up and shows the time. If the watch does not light up, it is switched off/discharged. You can turn it on again by holding down the alarm button, see description above. **The watch does not start automatically if it has discharged.**



Hold the alarm button until the Sensorem logo lights up to start the watch after it has been discharged.



- The watch may only be charged with the supplied charging cable and the wall adapter intended for the watch. It is not possible to plug the cable into, for example, a phone charger. This leads to the watch having a greatly reduced battery life, alternatively it cannot be charged at all.
- **Contact us if you need an extra charger or cable and we will send them out to you free of charge.**

5. How to press the alarm?

- The alarm can be activated regardless of if the watch is charging, is in standby mode or in the main menu. To activate the alarm, press and hold the alarm button for three seconds, then release the button. The user receives clear feedback that the alarm has been activated by the text "Ringer/I samtal" (calling/in conversation) appearing on the watch's screen, as well as the phone number below (Picture 6). **ATTENTION!** The phone number shown on the watch's screen during an alarm is always +46 8 400 747 66, which is the number to Sensorem's switchboard service. The switchboard service forwards the call to the right relative.



Picture 6

- 1.** All predefined relatives receive notifications on their mobile phones about the alarm, and the user's GPS position and status are updated in the Sensorem app. **ATTENTION!** The user's GPS position is always displayed in the app when an alarm has been set off but it is also updated semi-regularly when the watch is turned on. When updating the current position, the first position shown may circle a larger area. It is therefore important to wait 1 minute until the circle has decreased in size, for a more accurate position.
- 2.** At the same time, the watch calls the predefined relatives in the desired order, for example "Pelle", "Kalle" and "Lisa". If "Pelle" cannot receive the call, the watch automatically calls "Kalle" followed by "Lisa". If none of them answers, the watch will call all relatives a second time. If no one answers the second time, the call is forwarded to Sensorem's alarm center, which then acts upon the alarm (open 24/7).

6. To receive calls to the watch

- The welcome letter states which phone number the watch has been assigned. The phone number is a regular mobile phone number that you can receive calls from at any time.
- When someone is calling the watch, a melody loop is played on the device so that the wearer of the watch clearly can hear that someone is calling. The user answers the call by pressing the green button that clearly appears on the screen.



Recommendation

Make sure the watch is turned on. The functions of the watch do not work when the watch is switched off. The watch is also waterproof and can be worn everywhere (in the shower works fine, but do not swim with it).

7. To change the bracelet on the watch

- There are two types of bracelets for the watch, a rubber bracelet with clasp, and a braided fabric bracelet. These have two different types of attachments and are replaced as shown below:



The braided fabric bracelet has a fastener that is marked in the picture with a red circle. To remove the bracelet, pull the clasp downwards, which causes it to release from the watch. To put the bracelet on, the clasp is pulled down to insert and can then be released.



The rubber band/original band with snap requires a pin, or similar, to push the fastener down. A "click" results in the bracket releasing the watch.

8. Specifications and recycling

Size	49,5X42,5X15,8 cm
Battery capacity	850 mAh
Weight	50 g
Battery time / Standby time	2-3 days
IP Classification	IP67
Communication	4G/3G/2G
Position	GPS, AGPD, LBS, WIFI



The product must be disposed of separately from other household waste. Please dispose the product after the end of its useful life in a container intended for electronic equipment.

9. Contact information

Sensorem AB

Kaptensgatan 6
114 57 Stockholm



Email: support@sensorem.se

Tel: +46 8 46 00 77 00